

A black ergonomic chair with horizontal ribbing is shown from a side profile, partially cut off by the frame. The chair has a high backrest and a thick seat cushion. The background is white.

# OPTI CUSHIONS

OPHTALMIC CHAIR UPHOLSTERY SET

# INVENTORY SET

Set comes with the following items:



**MARCO OPHTHALMIC**  
CHAIR ENCORE 1280

## HEADREST



## BACKSEAT

Straps Attached




## LEGREST

Straps Attached



## SEAT

Straps Attached

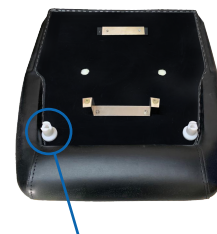

Straps attached

# ASSEMBLY INSTRUCTIONS

This drawing refers to MARCO Ophthalmic Chair Encore 1280

## STEP 1

### HEADREST

Unscrew your current Headrest. Put your new Headrest in same position as it was the previous one, screw back.

## STEP 2

### BACKSEAT

Remove with a strong pull your old backrest. With the straps aligned to the chair structure, push your new back rest until you hear and feel a "click".

## STEP 3


### LEGREST

Remove with a strong pull your old seat. With the straps aligned to the chair structure, push your new seat until you hear and feel a "click".

## STEP 4

### SEAT

Remove with a strong pull your old leg rest. With the straps aligned to the chair structure, push your new leg rest until you hear and feel a "click".

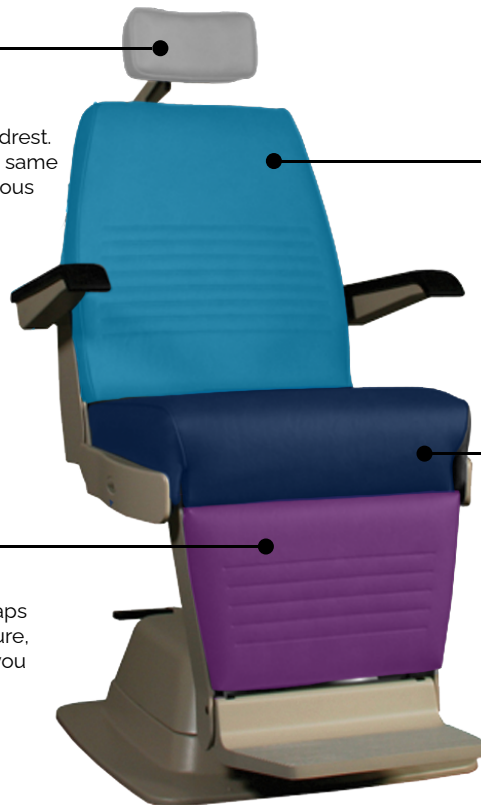

**MARCO OPTHALMIC**  
CHAIR ENCORE 1280



**OPTI**   
**CUSHIONS**