


OPTI CUSHIONS

OPHTHALMIC CHAIR UPHOLSTERY SETS

GROUP 1

HEADREST

2 Screws on the backside

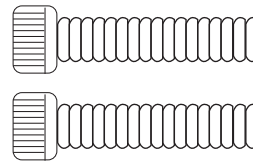


SET COMES WITH THE FOLLOWING ITEMS:

INVENTORY SET

GROUP 2

BACKSEAT


Velcro, no screws

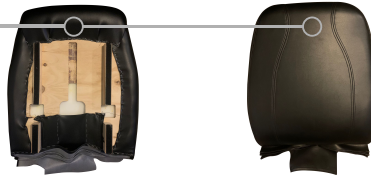

GROUP 3

ARMREST


Left and right. 3 screws on each armrest on backside.

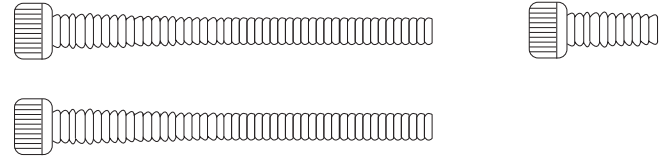

GROUP 4

LEGREST

2 screws on the backside

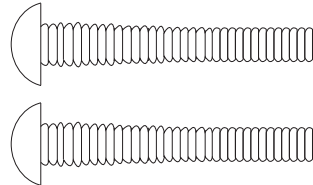

2 Hex Key Allen Wrench



Big 3/16

Small 5/32

GROUP 5

SEAT

Velcro, no screws

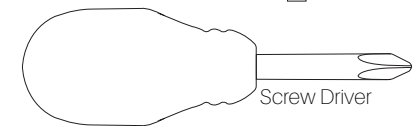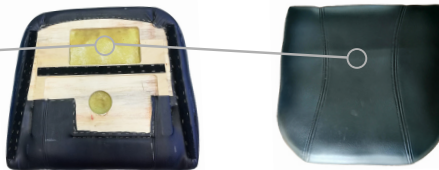


SCREW IMAGES ARE ON REAL SCALE

TOOLS**ASSEMBLY INSTRUCTIONS**

This drawing refers to TOPCON ophthalmic chair OC-2200

STEP 1**HEADREST**


Insert both screws in headrest backside from behind the chair structure and tighten screws with allen wrench.

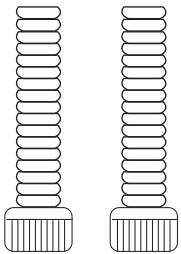**STEP 2****BACKSEAT**

Identify velcro guide on the backseat backside. Align with velcro on chair structure and press all edges until it fits perfectly with chair structure.

**STEP 3****ARMREST**


Insert and tighten first the two bigger screws on the elbow side and then the smaller screw on the hand side. Use the allen wrench.

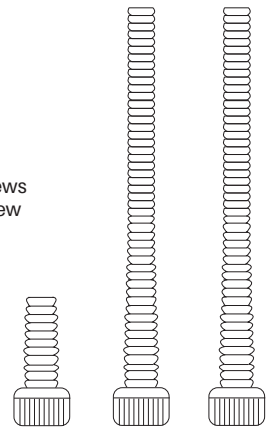**STEP 5****SEAT**

Identify velcro guide on seat backside, align with velcro on the chair structure and press to stick. Make sure to align edges.

STEP 4**LEGREST**

Insert first screw in the hole on top from behind the chair structure. Tighten the screw with allen wrench. Insert second screw in the hole on the bottom behind the chair structure. Tighten the screw with allen wrench.

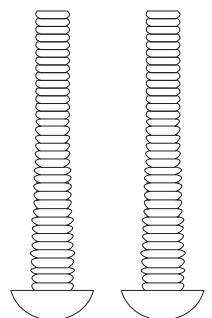

*Tip assembly: Tighten first all screws at half strength and then do a final tighten at full strenght.
 ** Set comes with an additional velcro so you can replace the one on the chair structure if needed. just make sure you cut it with a cutter and then add some hard glue on the chair structure side.